


PRO BATH CHAIR LIFT®

SET UP GUIDE



p3

10. Extend the mast by hand until snug; further tighten 2 revolutions with a 1¼" wrench; tighten down the locking nut.

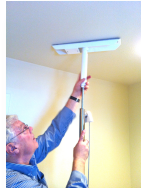

11. Drill two 1/2" holes through the holes in the mast head plate and install two heavy duty 3/16" SNAPTOGGLE® Toggle Bolts provided. *See Below



12. Install two stainless screws in the foot plate.
13. Install the safety bar, adjusted for the comfort of the user.

Please direct any questions or inquiries to our Customer Service Department @ 1-800-729-6216

* Installation Details For The TOGGLER® Brand SNAPTOGGLE® Bolts can be found on the following page. The bolts are available at LOWES®.

(C) 2013 Safe bathtub, Inc., All rights reserved.

All photos and descriptions herein are copyrighted by or under license to Safe Bathtub, Inc., and are not to be reproduced except by written permission.