



# PRO BATH CHAIR LIFT®

## SET UP GUIDE p2

\*Note: If you have a jacuzzi tub or a tub with a TALL tub wall, you will use an extended arm (14" or more); Rotate the foot of the mast 5 degrees; this allows the chair back to clear the tub wall on the outside of the tub, allowing it to be lowered nearly to the floor.

### STEP 2 ASSEMBLE AND SET UP

1. Measure floor to ceiling & adjust the threaded mast extension to 1" less than the floor to ceiling height; slide on the cover shield.
2. Attach the chair to the slider with the 1/4"x 2" SS bolts & nuts provided - left or right as needed.



3. Install the mast into the chair slider with the mast sheave toward the tub. Place the mast head plate on top of the mast.
4. Set the power lifter parallel to the front wall of the tub, with the mast stud facing the tub drain. The mast should be at the high end of the tub floor.
5. Set the mast and chair slider on the mast base stud; the chair can rest on the tub wall.
6. Remove the mast sheave and guard, and replace it with the cable installed; TIGHTEN the sheave and attach the cable to the chair lift.
7. Extend the mast to touch the ceiling; install the battery; lift the chair into the tub; center the chair on the tub floor.
8. With a level, straighten the mast until it is perpendicular.
9. Check the chair to be sure it is centered in the tub.