



PRO BATH CHAIR LIFT®

SET-UP GUIDE



1. Power Lifter
2. Mast
3. Upper Mast Extension with Screw and Lock
4. Upper Mast Adjustment Beauty Cover
5. Upper Mast Head Plate
6. Two SNAPTOGGLE® Toggle Bolts for Head Plate
7. Mast Sheave Cover
8. Mast Sheave
9. Mast Sheave Shoulder Bolt
10. Battery
11. Battery Charger
- 12a. Peri-Care Chair or 12b. Standard Chair (see below)
13. Chair Slider
14. Chair Safety Bar

Tools You Will Need:

- Tape Measure
- 9/16" Open End Wrench
- 2 Allen Wrenches: 3/16" 5/16"
- Adjustable End Wrench up to 1 1/4"
- Hand Drill
- A 1/2" Bit
- Phillips Screwdriver
- Level

Time You Will Need:

Approximately 90 Minutes

STEP 1 MEASURE

- A. Bottom of Tub Width - Minimum 18"
- B. Floor to Top of Tub Wall - Maximum 24" (Call for extensions)
- C. Standard Ceiling Height 88" - 96"
Exceptions require factory alterations; call 1-800-729-6216
- D. Width of PRO Chair and Lift Mechanism 43" minimum
- E. Clearance Needed OUTWARD from the Front Tub Wall 12"
- F. Radius of swing 27" or 32"



NOTE: Some installations may require rotating the toilet 45 degrees and refastening it to the floor to gain the space needed for proper operation of the Safe Bathtub Inc Pro Bath Chair Lift®

